
# Surface Disinfectant Wipes

**For the surface disinfection of non-invasive medical devices.**  
Alcohol is not recommended for use on dental/medical instruments.

**For the disinfection of surfaces. Use biocides safely and sustainably.**  
This product should be used in accordance with the product label.  
PT2 Formulation type: AL.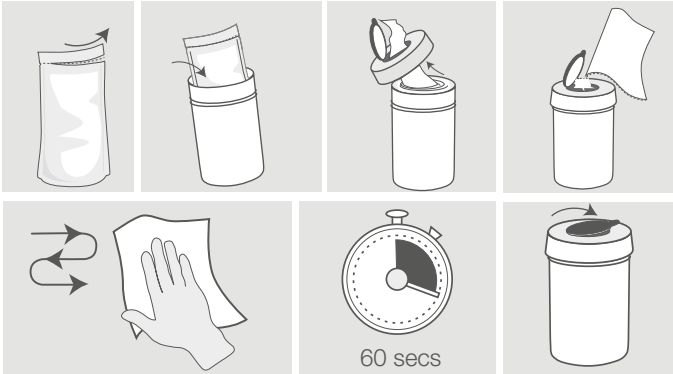

**Directions for use:** 1. Remove cap of dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficed wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action for 60 seconds. Allow surface moisture to evaporate. For professional use only.

**Efficacy (60 secs):** Bactericidal including MRSA, VRE. Yeastcidal. Mycobactericidal. Tuberculocidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

**Tested to Standards:** EN 16615, EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

**Composition:** Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes.

**Danger:** H225: Highly flammable liquid and vapour. H319: Causes serious eye irritation. **First aid procedures.** P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **Fire procedures.** P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. **Storage.** P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Incompatible materials: Copper, aluminium. Do not use on surfaces sensitive to alcohol. **Disposal.** Dispose of contents/container in accordance with local regulations. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

## Efficacy

Microorganism	Time
<b>Bactericidal</b>	<b>60 sec</b>
Pseudomonas aeruginosa	60 sec
Staphylococcus aureus	60 sec
Enterococcus hirae	60 sec
E. coli	60 sec
MRSA	60 sec
VRE	60 sec
<b>Yeastcidal</b>	<b>60 sec</b>
Candida albicans	60 sec
<b>Enveloped viruses</b>	<b>60 sec</b>
Hepatitis B virus (HBV)	60 sec
Hepatitis C virus (HCV)	60 sec
HIV	60 sec
Measles virus	60 sec
Hepatitis Delta virus	60 sec
Influenza virus	60 sec
Coronavirus	60 sec
Herpesviridae	60 sec
Filoviridae (Ebola)	60 sec
Rubella virus	60 sec
Rabies virus	60 sec
Poxviridae	60 sec
<b>Non enveloped viruses</b>	<b>60 sec</b>
Adenovirus 5	60 sec
Murine norovirus	60 sec
<b>Mycobactericidal</b>	<b>60 sec</b>
Mycobacterium avium	60 sec
<b>Tuberculocidal</b>	<b>60 sec</b>
Mycobacterium terrae	60 sec

UK	Danger.
BG	Опасно.
CZ	Nebezpečí.
DE	Gefahr.
DK	Fare.
EE	Ettevaatus.
ES	Peligro.
FI	Vaara.
FR	Danger.
GR	Κίνδυνος.
HR	Opasnost.
HU	Veszély.
IT	Pericolo.
LT	Pavojus.
LV	Bīstami.
NL	Gevaar.
NO	Fare.
PL	Niebezpieczeństwo.
PT	Perigo.
RO	Pericol.
SE	Fara.
SI	Nevarno.
SK	Nebezpečnost.

Lemon UFI:EE00-V0C9-F00M-WVUD  
Natural UFI:HN00-D01P-S004-K6YW

Manufactured in the UK by:  
Topdental (Products) Ltd.  
12 Ryefield Way, Silsden,  
W. Yorks BD20 0EF England, UK.  
Tel +44 (0)1535 652 750  
www.topdental.co.uk

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